



## APTIMA Urine Specimen Collection Kit for Male and Female Urine Specimens

For *in vitro* diagnostic use.

For US export only.

### Intended Use

The APTIMA Urine Specimen Collection Kit for Male and Female Urine Specimens is for use with APTIMA assays. The APTIMA Urine Specimen Collection Kit is intended to be used for the collection and transport of male or female urine specimens.

### Materials Provided

50 APTIMA Urine Specimen Collection Kits for Male and Female Urine Specimens (Cat. No. 301040)

Each kit contains:

Component	Quantity	Description
Pipette	1	Disposable transfer pipette.
Specimen transport tube	1	Tube containing 2.0 mL APTIMA urine transport medium.

### Warnings and Precautions

- A. Do not apply the transport medium directly to skin or mucous membranes or take internally.

### Kit Storage Requirements

Store collection kit at room temperature (15°C to 30°C).

### Urine Specimen Performance

The assay performance characteristics of the male and female urine specimens are provided in the appropriate APTIMA assay package insert. The APTIMA assay package inserts may be referenced online at [www.gen-probe.com](http://www.gen-probe.com). The performance of the male urine specimen has not been established for all APTIMA assays. The table below identifies the acceptable specimen types for each of the APTIMA assays.

APTIMA Assay for	Female Urine Specimens	Male Urine Specimens
<i>Chlamydia trachomatis</i>		
<i>Chlamydia trachomatis</i> and <i>Neisseria gonorrhoeae</i> (APTIMA COMBO 2 Assay)	Yes	Yes
<i>Neisseria gonorrhoeae</i>		
<i>Trichomonas vaginalis</i>	Yes	No

### Specimen Collection and Handling

1. The patient should not have urinated for at least 1 hour prior to specimen collection.
2. Direct patient to provide a first-catch urine (approximately 20 to 30 mL of the initial urine stream) into a urine collection cup free of any preservatives. Collection of larger volumes of urine may result in rRNA target dilution that may reduce test sensitivity. Female patients should not cleanse the labial area prior to providing the specimen.

3. Remove the cap and transfer 2 mL of urine into the urine specimen transport tube using the disposable pipette provided. The correct volume of urine has been added when the fluid level is between the black fill lines on the urine specimen transport tube label.
4. Re-cap the urine specimen transport tube tightly. This is now known as the processed urine specimen.

## Specimen Transport and Storage

After collection, transport the processed urine specimens in the APTIMA urine specimen transport tube at 2°C to 30°C and store at 2°C to 30°C until tested. Processed urine specimens should be assayed with the APTIMA assay within 30 days of collection. If longer storage is needed, refer to the appropriate APTIMA assay package insert.

Urine samples that are still in the primary collection container must be transported to the lab at 2°C to 30°C. Transfer the urine sample into the APTIMA urine specimen transport tube within 24 hours of collection. Store at 2°C to 30°C and test within 30 days of collection.

**Note:** Specimens must be shipped in accordance with applicable national and international transportation regulations.

## Limitations

- A. Use this collection kit only with the APTIMA assays. Performance has not been established with other products.
- B. The performance of male urine specimens has not been established for the APTIMA *Trichomonas vaginalis* assay.



Gen-Probe Incorporated  
San Diego, CA 92121 USA

### U.S. and international contact information

Customer Service: +1 858 410 8002  
customerservice@gen-probe.com  
Technical Support: +1 858 410 8511  
technicalsupport@gen-probe.com

### Toll-free from U.S. and Canada

Customer Service: +1 800 523 5001  
Technical Support: +1 888 484 4747

[www.gen-probe.com](http://www.gen-probe.com)

**EC** **REP**

EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands

GEN-PROBE, GEN-PROBE and design, APTIMA, APTIMA and design, and APTIMA COMBO 2 are trademarks of Gen-Probe Incorporated.

© 2001-2012 Gen-Probe Incorporated  
502136EN Rev. B  
2012-11